


# Quick Reference Guide

## Instructions for putting the Midmark Digital Spirometer in PCP mode

- 01** Click on the **Settings** button from the IQmanager® main screen and navigate to the Devices tab.
- 02** Select the **Spirometry Settings** button.
- 03** Under the Configuration section, select **PCP Mode** from the Configuration Profile dropdown.
- 04** Click the **Save** button on the bottom left hand side of the screen. This will save the setting change and return you to the IQmanager Configurations Settings screen.
- 05** Click the **Back Arrow** in the Configuration Settings screen. This will take you back to the IQmanager Opening Screen.
- 06** **Search** for a patient or select **New Patient** to start a spirometry test.

# Interpretation Decision Tree

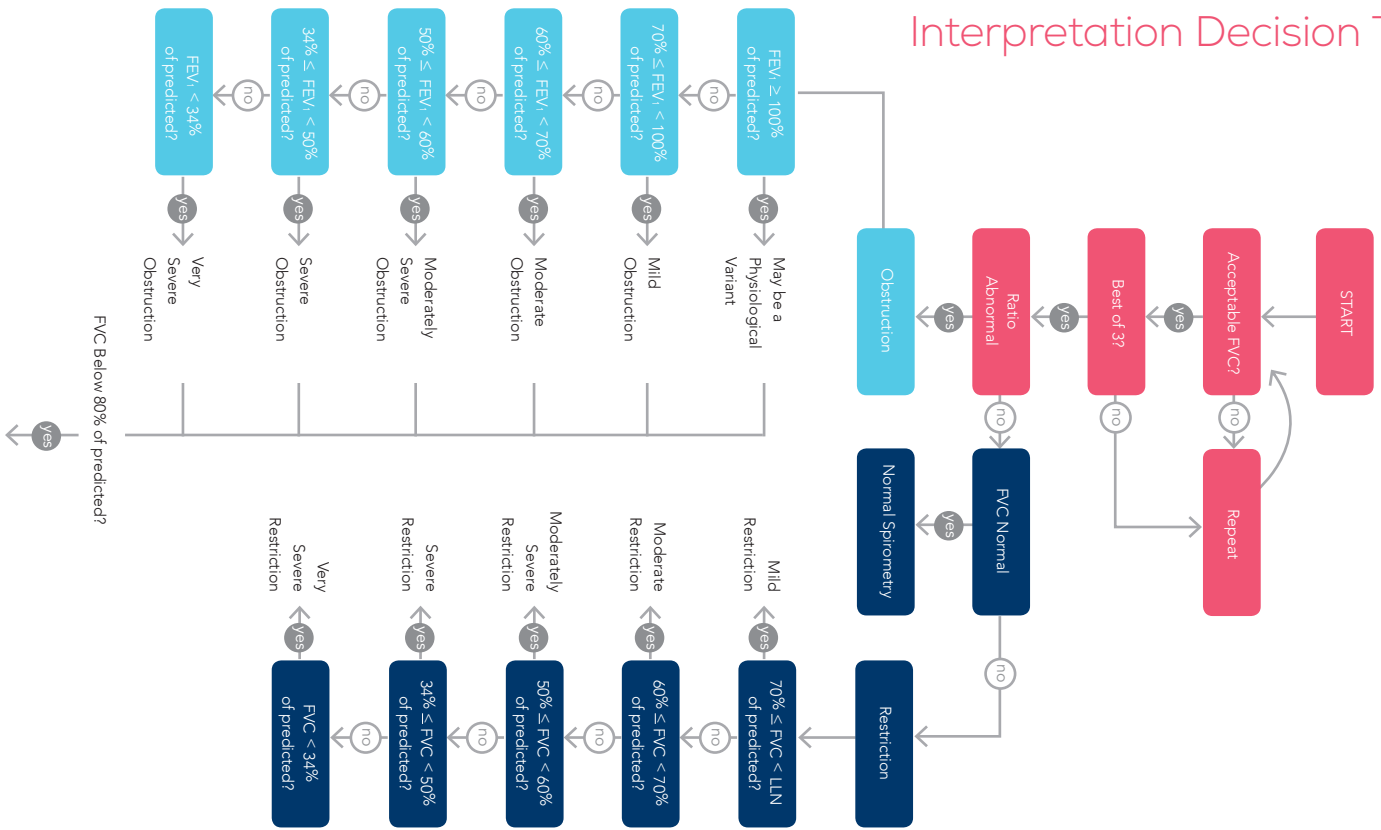

And low vital capacity, cannot rule out superimposed restriction.