



# 12:00 Procenter Delivery Mounting Kit Installation Instructions [002-10005-00]


**Components:**  
Mounting Plate..... 1  
Slide..... 2  
Template..... 1

**Special Tools:**  
Router

**Note**


The following instructions are recommendations for mounting the 12:00 Procenter delivery unit(s) onto non-Midmark cabinetry, laminate particle board, cabinet grade plywood or wood mounting surfaces approximately 3/4 inches thick or thicker. Refer to instructions from delivery unit or related components for any remaining installation.

Slide (2;  
for delivery  
unit)

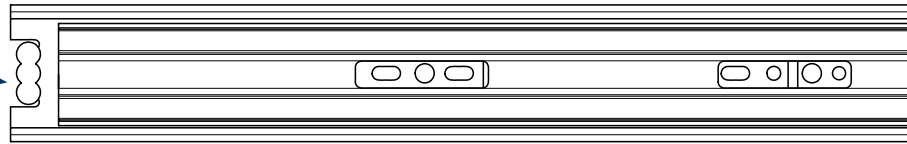

Slide Template  
(003-10008-00)

Mounting  
Plate

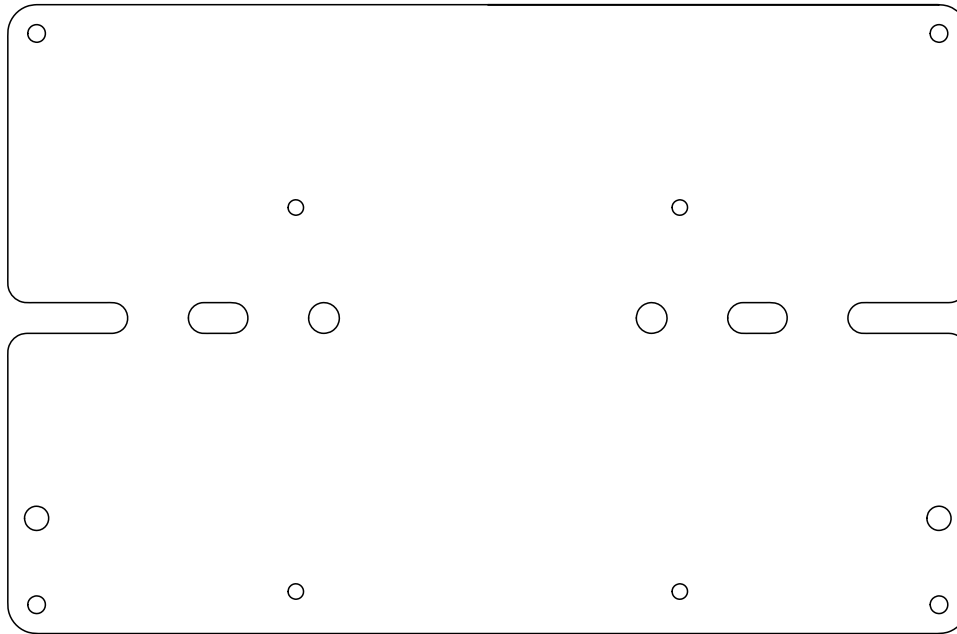


DA4868i

# Slide Installation

**Step 1: Install two slides to cabinet interior.**

- A) Using provided template, drill six 3/32 inch pilot hole locations for cabinet members as shown.
- B) Secure slides to cabinet using four #8 x 3/4 inch screws (not included) in locations shown.

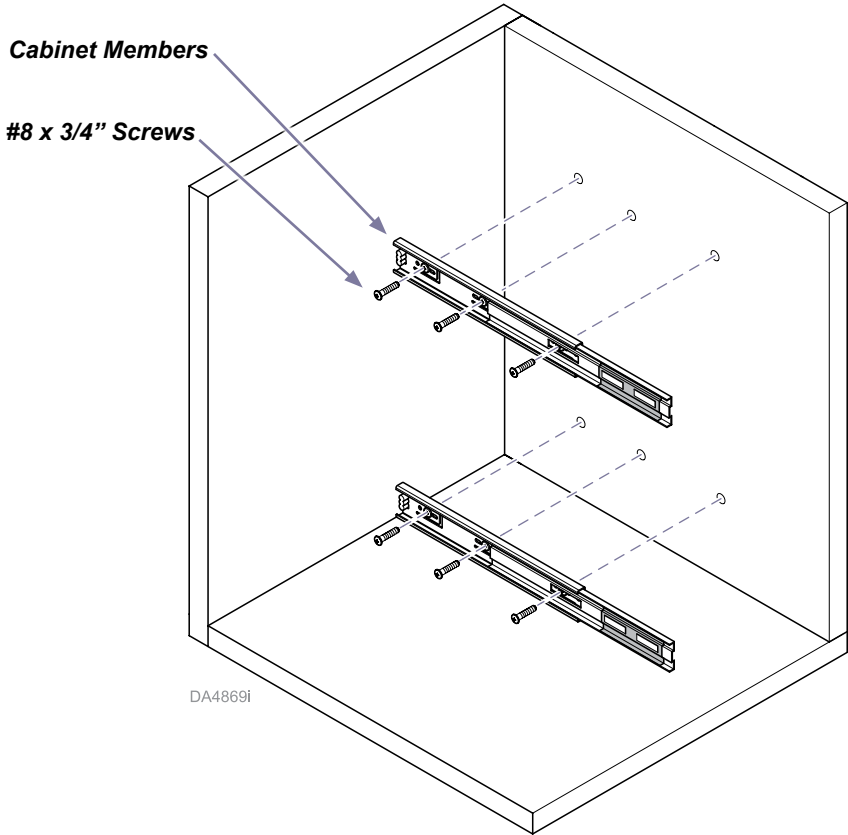

# Mounting Plate Installation



## Equipment Alert

When laying out the mounting plate inlay, orient as shown and ensure plate aligns with vertical partitions (plate holes for wood screws) to correctly secure mounting plate.




## Caution

Complete routing work in a safe, clear area. Mounting plate must be oriented as shown, otherwise holes will misalign.

## Step 2: Route mounting area.

- A) Using plate as guide, measure area to route on top surface.
- B) Route at 1/4 inch depth so plate sits flush with top surface.

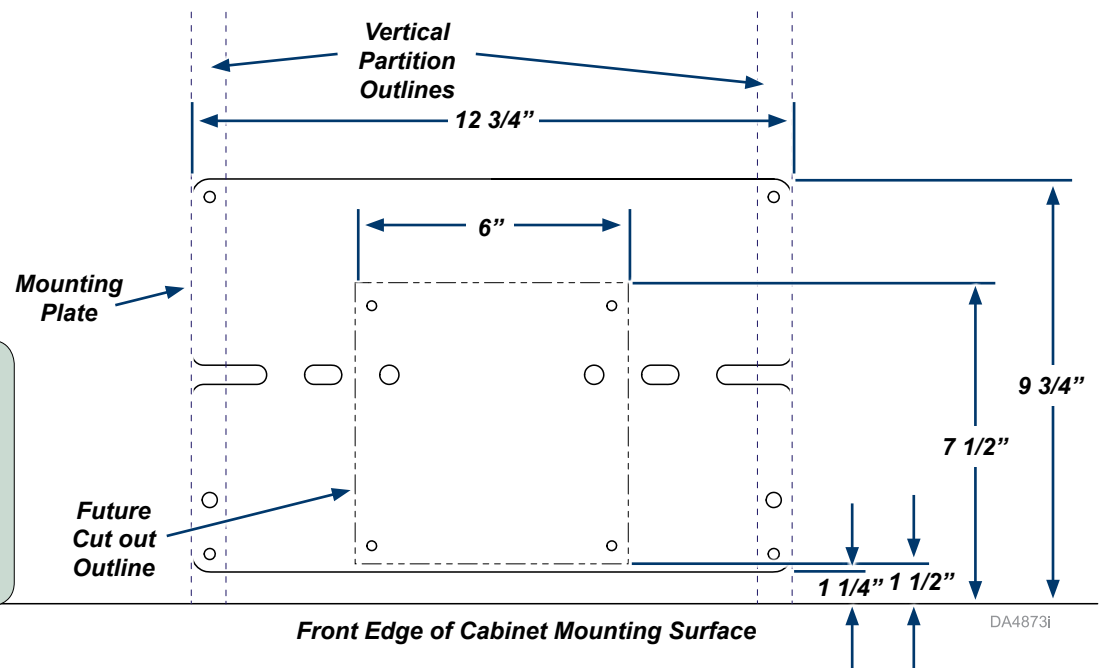


## Caution

Complete cut out work in a safe, clear area.



## Step 3: Cut out square in top surface.

- A) Cut out a square area to expose the mounting plate underneath.

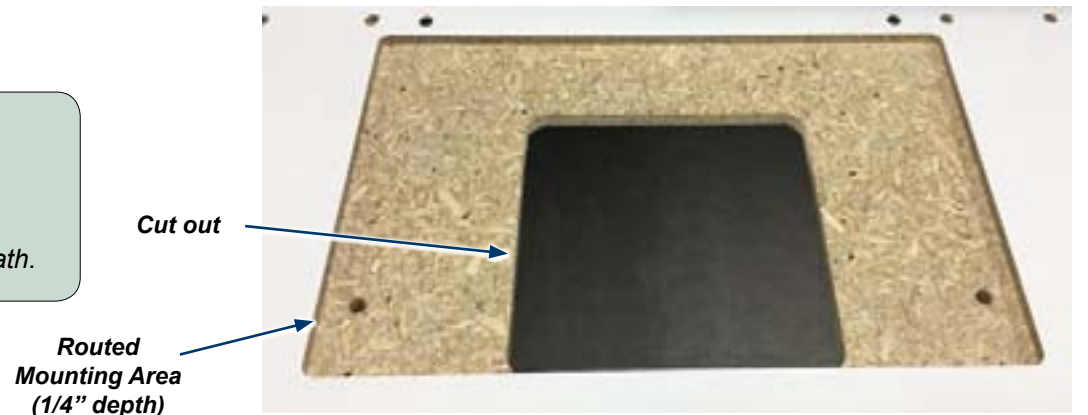

**Step 4: Install mounting plate to routed area in top surface.**

- A) Place plate in routed area in top so cut out area is visible as shown.
- B) Secure plate with four wood screws with 1-1/4" length in locations shown.

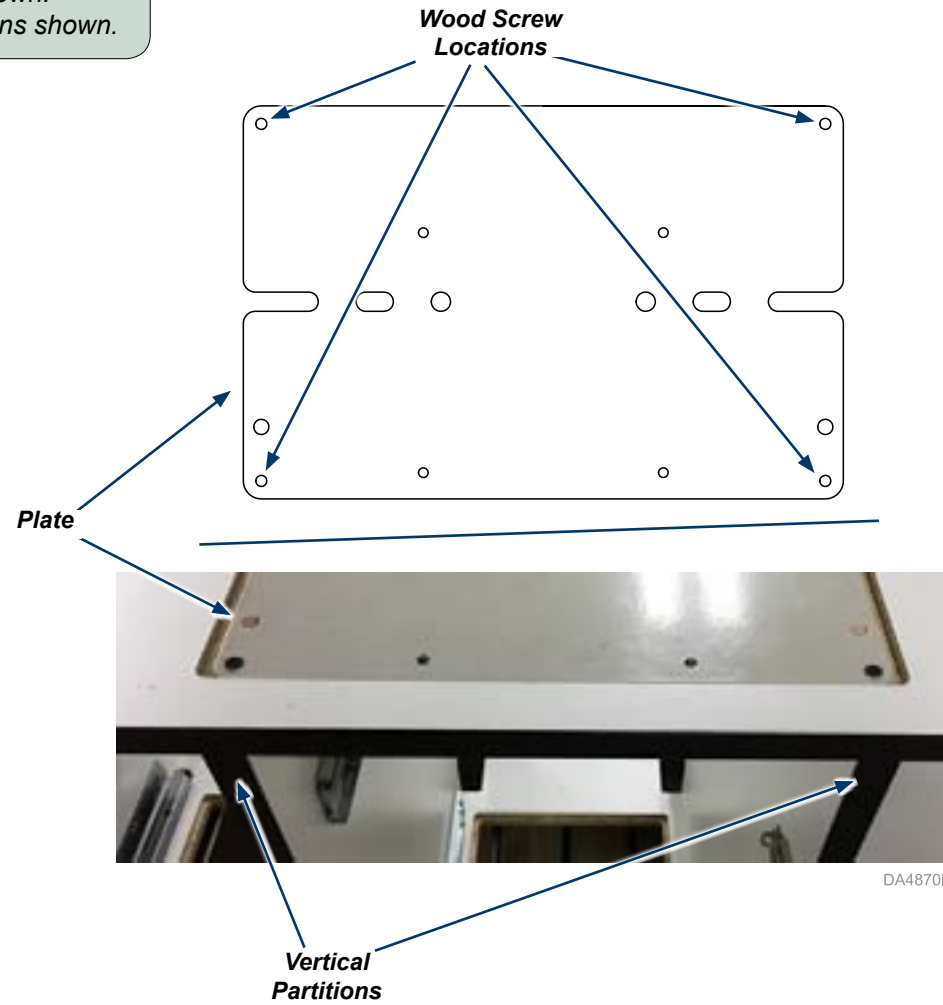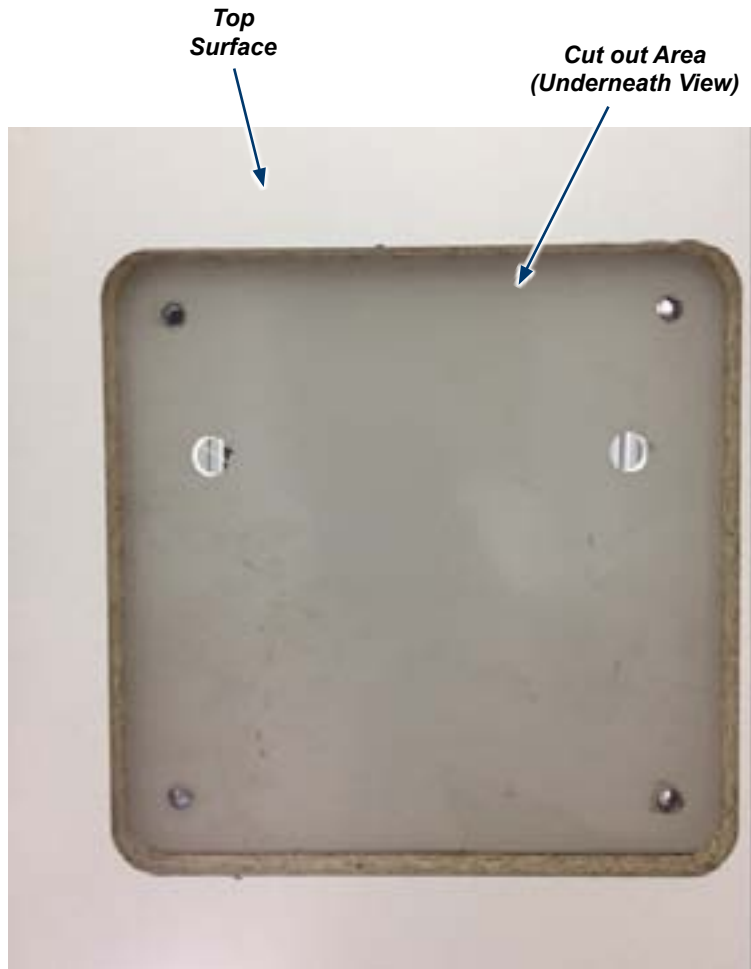

**Step 5: Finalize installation.**

- A) Refer to instructions from delivery unit or related components for any remaining installation.

