

Installation Instructions for

153941 (002-2018-XX) Power Supply Fuse Field Installation Kit

P/Ns 002-10131-00 (115V) and 002-10132-00 (230V)



WARNING: Always disconnect the power supply from the power source before removing any covers. Failure to do so may result in personal injury.

Step 1: Disconnect the power supply from the power source by disconnecting the power cord.

Step 2: Remove the single Phillips head screw and lock washer that are retaining the fuse access cover. Retain the screw and lock washer for reuse.

Step 3: Remove the cover and discard.

Step 4: Locate fuse labelled 'F8' on the printed circuit board.

Step 5: Remove fuse 'F8' taking care not to bend the fuse holder clips. Discard the fuse.

Step 6: Install the fuse, contained in the kit, in the location of the removed fuse.

Step 7: The power supply cover has an updated fuse label. Take the cover provided in the kit and install on the power supply.

Step 8: Install the screw and lock washer and tighten to secure the cover.

Step 9: Connect the power supply to the power source by reconnecting the power cord.