

Sopro Intraoral Camera Accessory Installation Guide on a 12:00 O'Clock Procenter Delivery Unit

Applies to Models:
153831-014

Special Tools:
none

Note

This installation guide is for installing a Sopro camera and docking station on a 12:00 Procenter delivery unit.



WARNING

Always disconnect power source before removing any covers. Failure to do so may result in personal injury.



Equipment Alert


The Sopro camera accessory is manufactured by ACTEON North America. Refer to ACTEON's literature for detailed information on Operation, Maintenance, Service, Troubleshooting, Warranty and to order replacement parts.

For additional information go to: <http://www.acteonsupport.com/>

Step 1: Remove control panel.

- A) *Open cabinet doors and clear space for access.*
- B) *Remove control panel from top and bottom slides.*

Note: Remove control panel carefully to avoid damage.

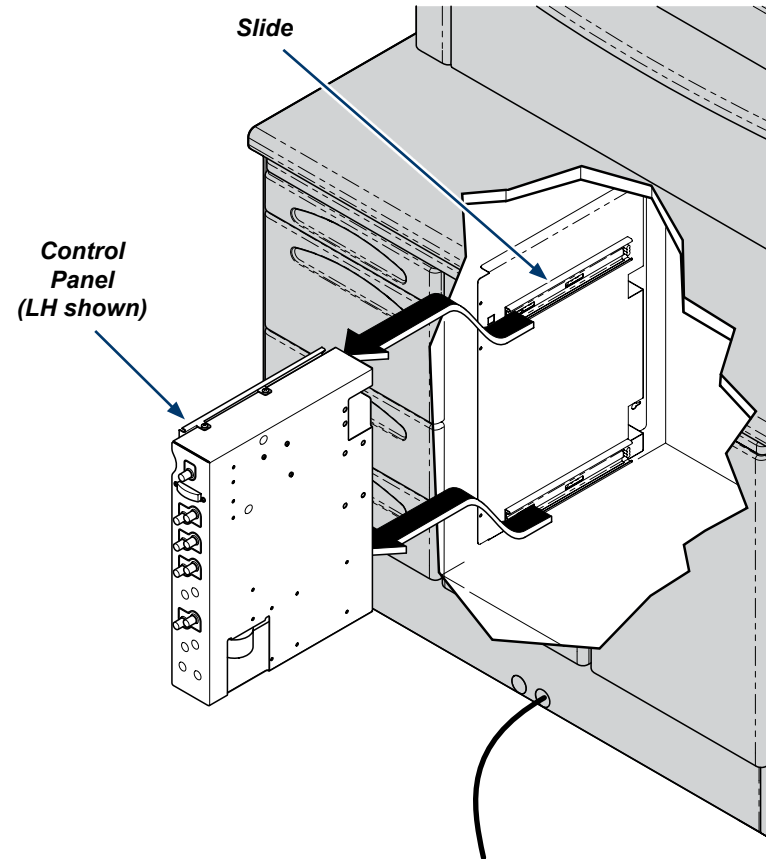

DA3855i

