

Procenter Cuspidor LR Mtd. Installation





Parts Bag Contents

- T-Fitting w/tubing 1
- Screw 2
- Lockwasher 2
- Cable Tie 2
- Clamp 1

Special Tools

Level

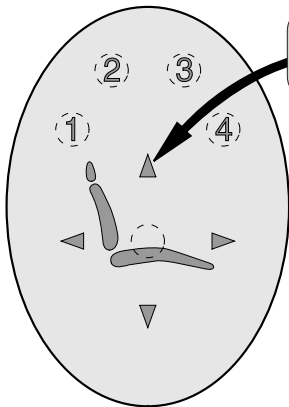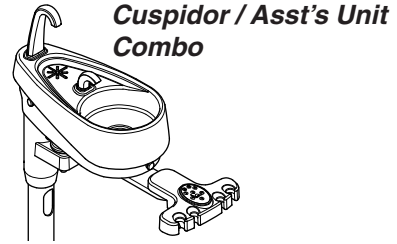


Step 1: Raise chair all the way up; then unplug the chair.

Step 3: Remove small cover.

Step 2: Remove safety bail cover.
Tip: Push cover IN & DOWN, then pull cover OUT & UP.

Step 4: Remove end cover.

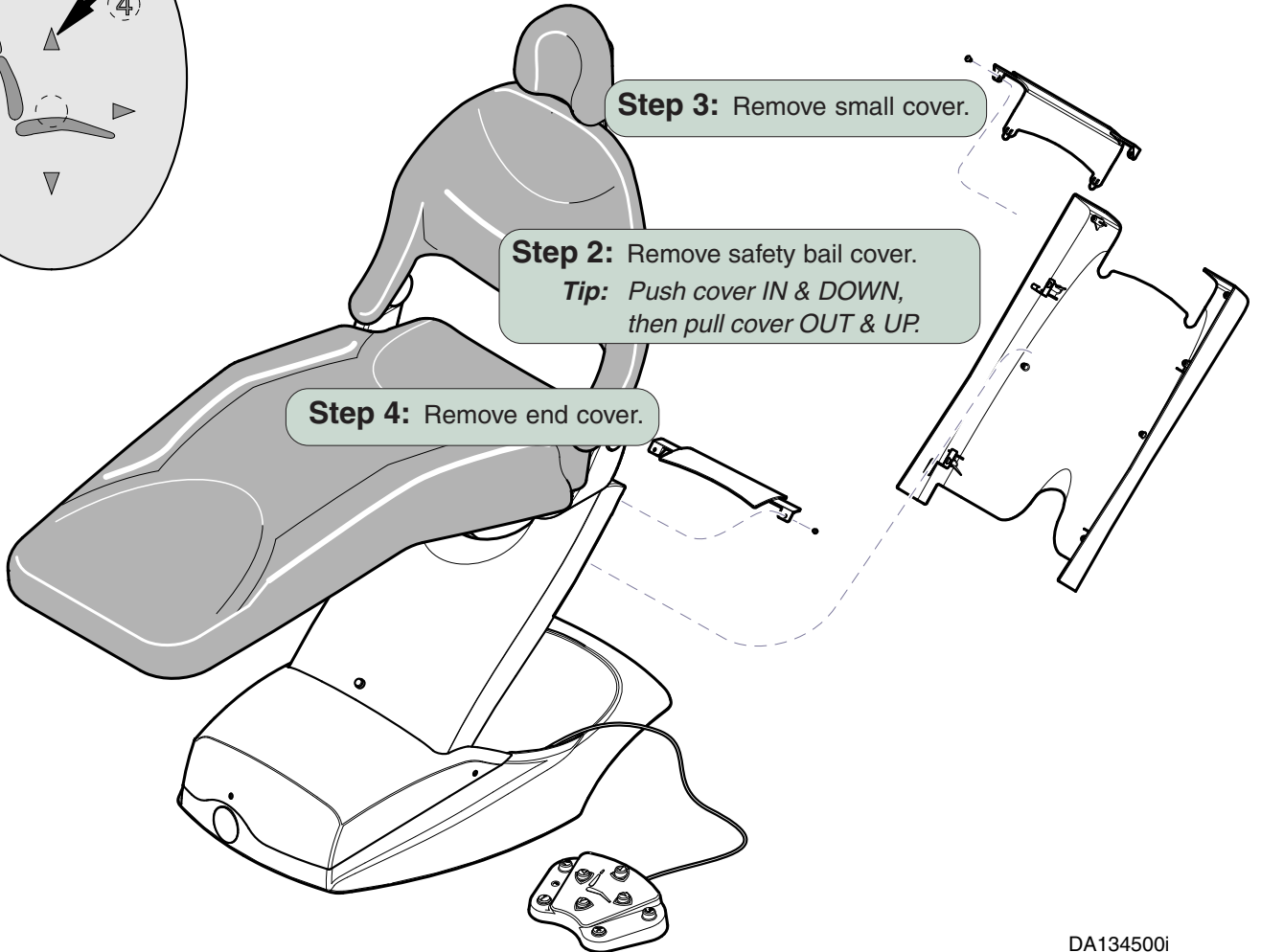


DA134500i



Equipment Alert

Have an assistant secure the cuspidor arm at all pivot points to prevent damage to equipment.

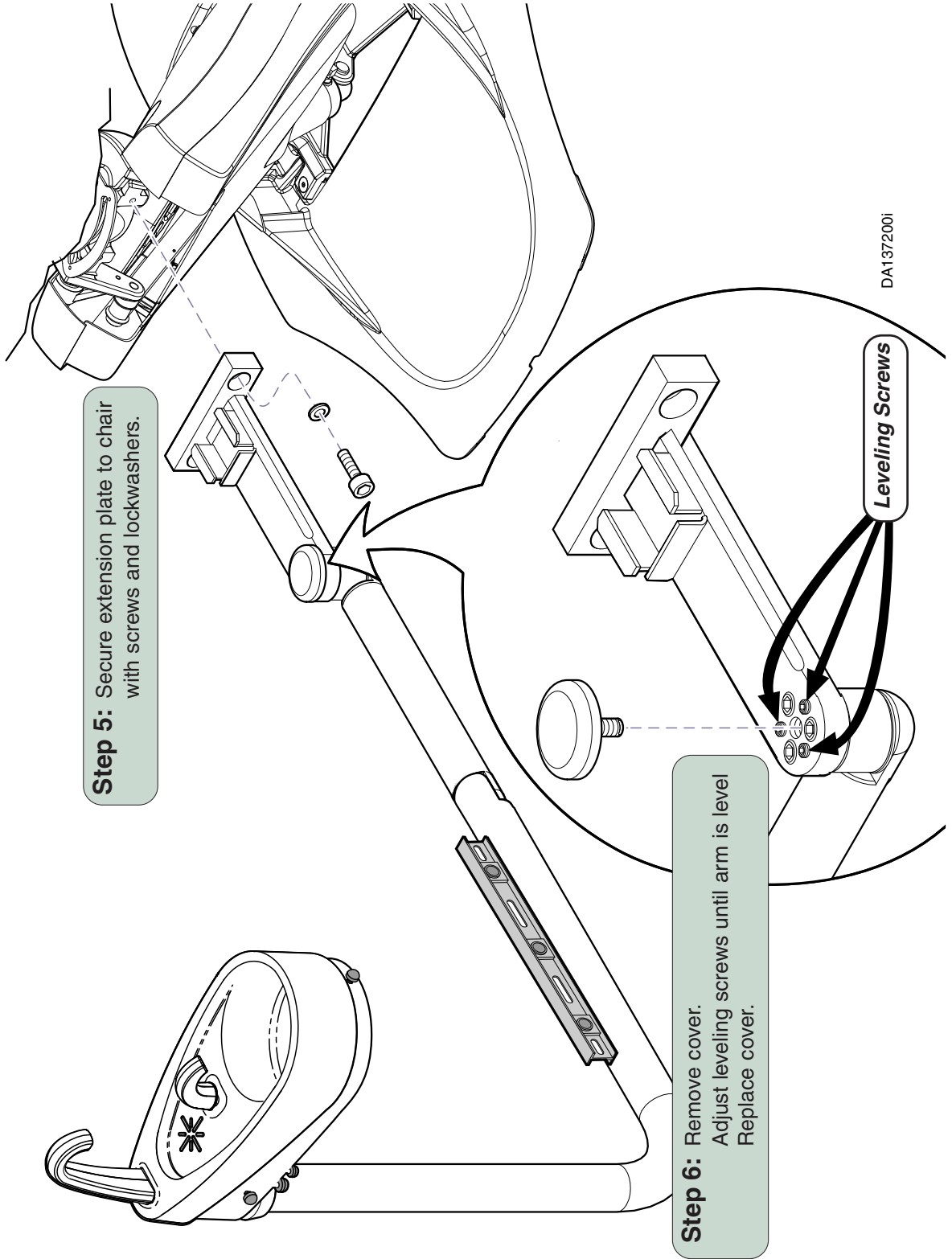


Step 5: Secure extension plate to chair with screws and lockwashers.

Step 6: Remove cover.
Adjust leveling screws until arm is level
Replace cover.

Leveling Screws

DA1372001

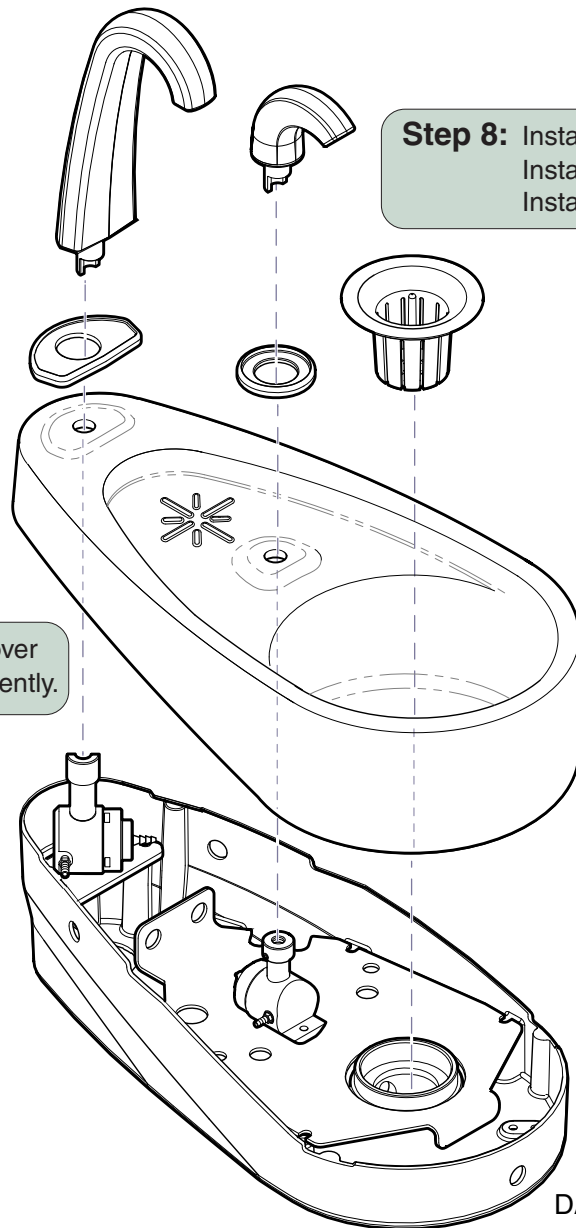

Step 8: Install faucet and gasket.
 Install bowl flush spout and gasket.
 Install gold trap.

Step 7: Position cuspidor bowl over base then press down gently.


DA137300i

Attention!

After completing Step 8:



***Standard Cuspidor,
 Go to page 4.***

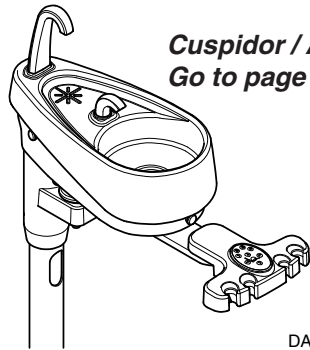

***Cuspidor / Asst's Unit Combo,
 Go to page 5.***


DA144700i

Attention!

This page applies to
Standard Cuspidor only.




For Cuspidor / Asst's Unit Combo,
Go to page 5.

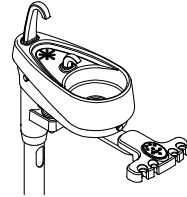

Standard Cuspidor


Step A: Pull out yellow tubing & cut.
Insert T-fitting w/yellow tubing.

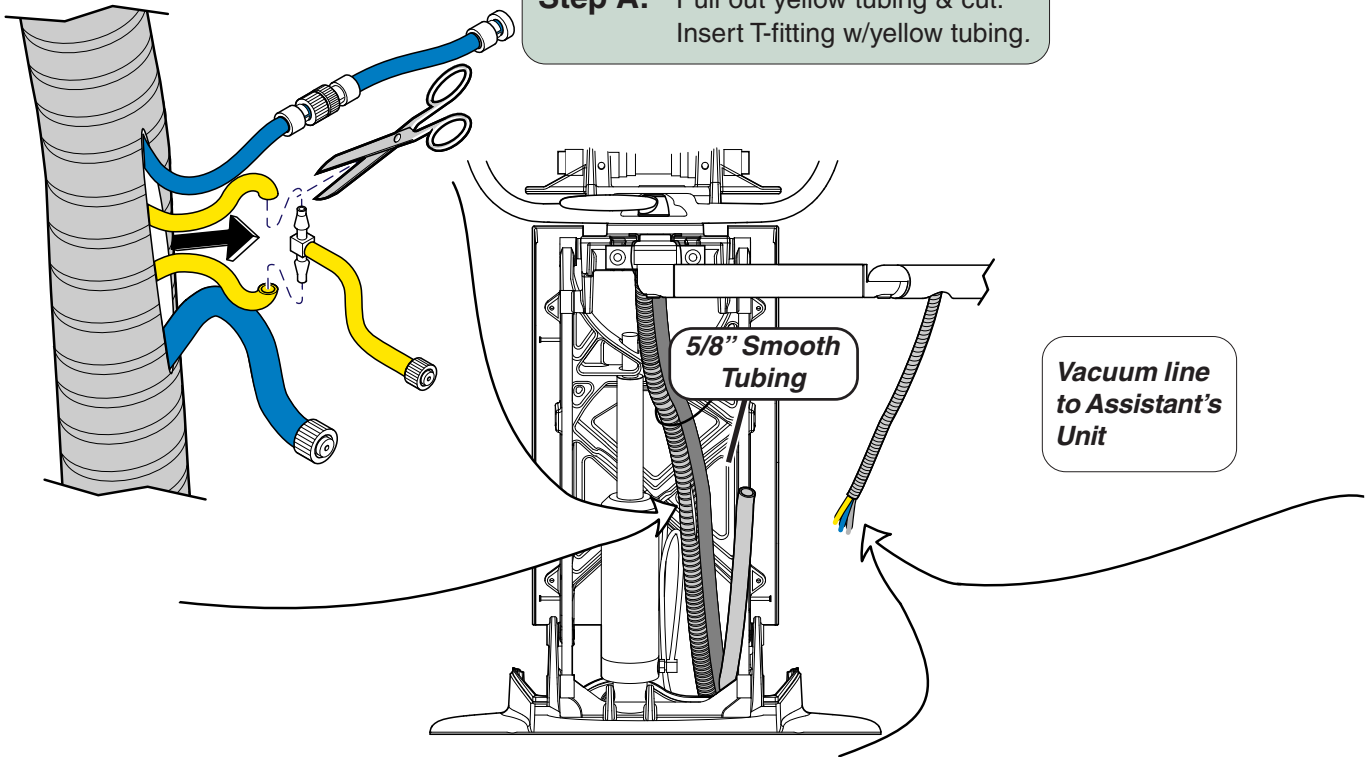

Standard Cuspidor

Step B: Connect 1/8" yellow tubing.
Connect 1/4" blue tubing.

Note: 1/8" blue tubing is not used.

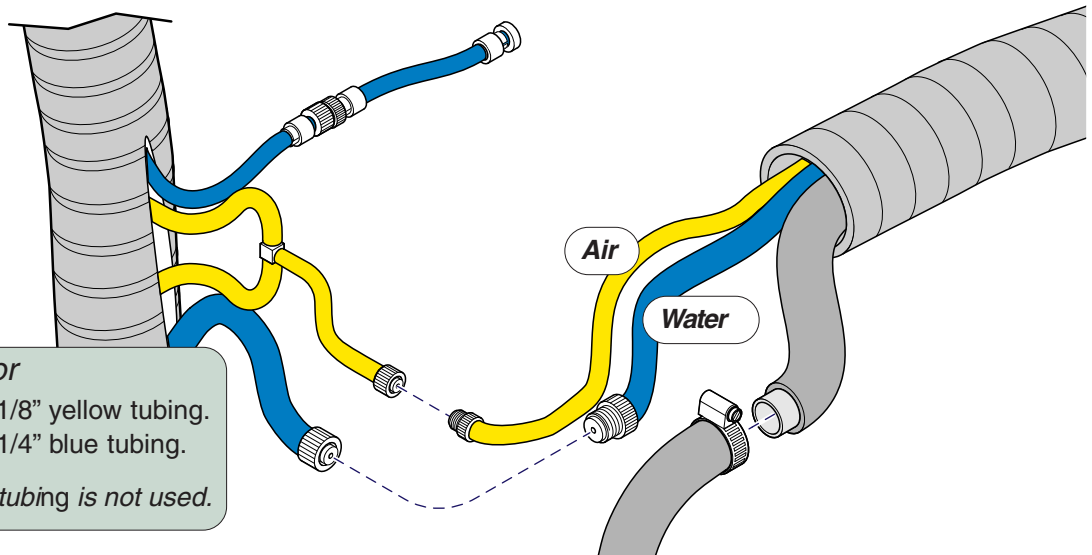

Standard Cuspidor


Step C: Connect 5/8" grey tubing from cuspidor
to 5/8" **smooth** tubing from umbilical.

Note: 5/8" **ribbed** tubing is for optional assistant's unit.

Go to Page 6

Attention!

This page applies to Cuspidor / Asst's Unit Combo only.

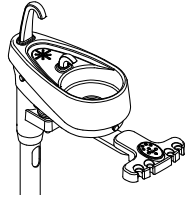

For Standard Cuspidor, Go to page 4.



Cuspidor/Asst's Combo

Step A: Remove plugged section from 1/8" blue tubing.

Cuspidor/Asst's Combo

Step B: Pull out 1/8" yellow tubing & cut. Insert T-fitting w/yellow tubing.

Vacuum line to Assistant's Unit

Vacuum line to console

Cuspidor/Asst's Combo

Step C: Connect 1/8" blue tubing. Connect 1/8" yellow tubing. Connect 1/4" blue tubing.

Cuspidor/Asst's Combo

Step D: Connect cuspidor drain tube to 5/8" smooth tubing from umbilical.

Water

Air

Water


Cuspidor/Asst's Combo

Step E: Connect vacuum tube to 5/8" ribbed tubing from umbilical.

DA3726i

Go to Page 6

Step 9: Remove plug from water shut-off valve.
Install 1/4" barb fitting.
Connect 1/4" blue tubing from umbilical.

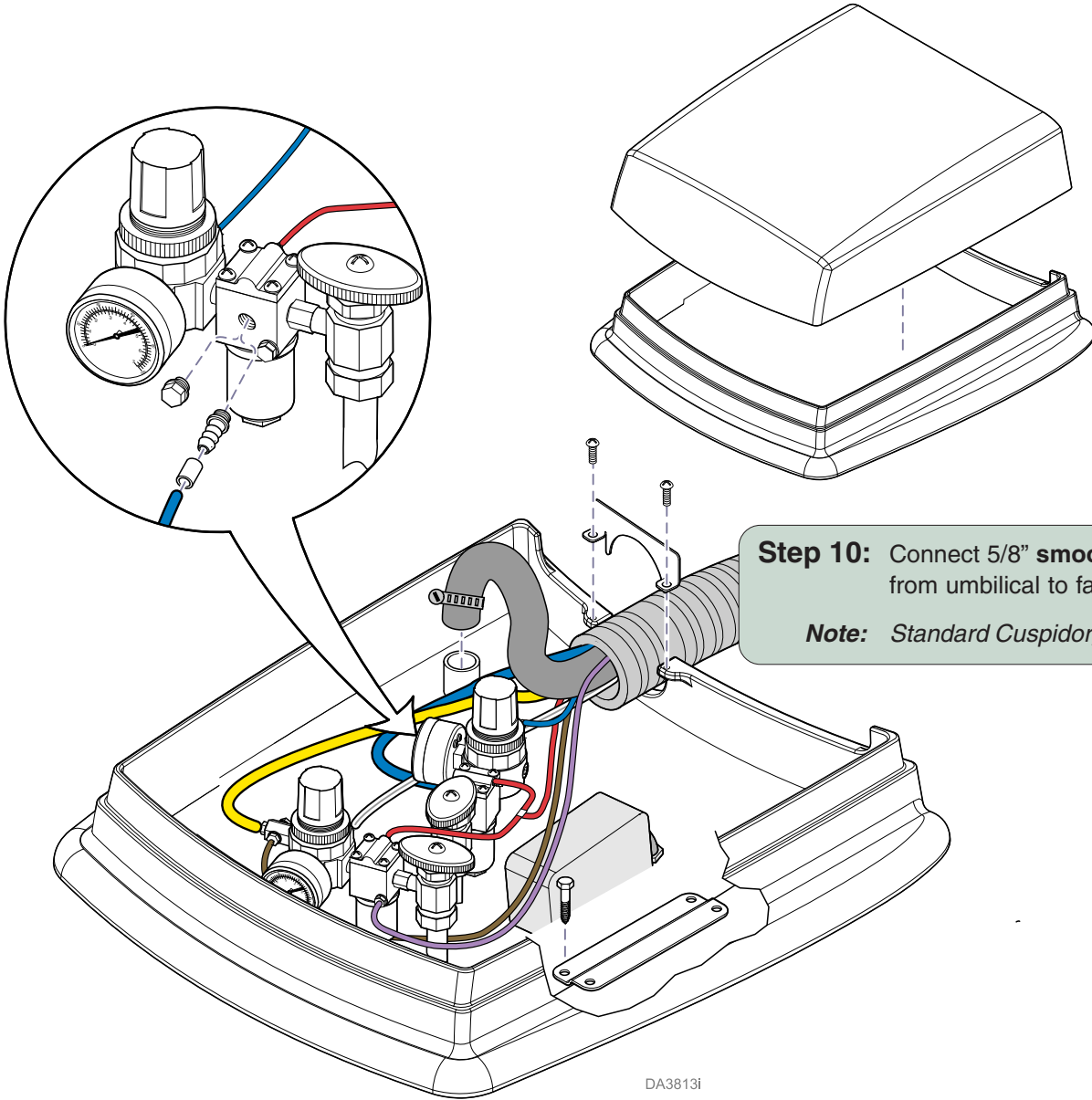

Step 10: Connect 5/8" **smooth** tubing
from umbilical to facility drain.

Note: *Standard Cuspidor, go to page 7.*

DA3813i

Cuspidor/Asst's Combo *only*:


Step 11: Connect 5/8" **ribbed** tubing (*not shown*)
from umbilical to central vacuum unit.





Caution

Connect ground wire to ensure system ground continuity. Failure to do so may result in electric shock.

Step 12: Connect wiring. Use two or more ties to bundle wiring and vacuum tubing together and attached to the chair frame.

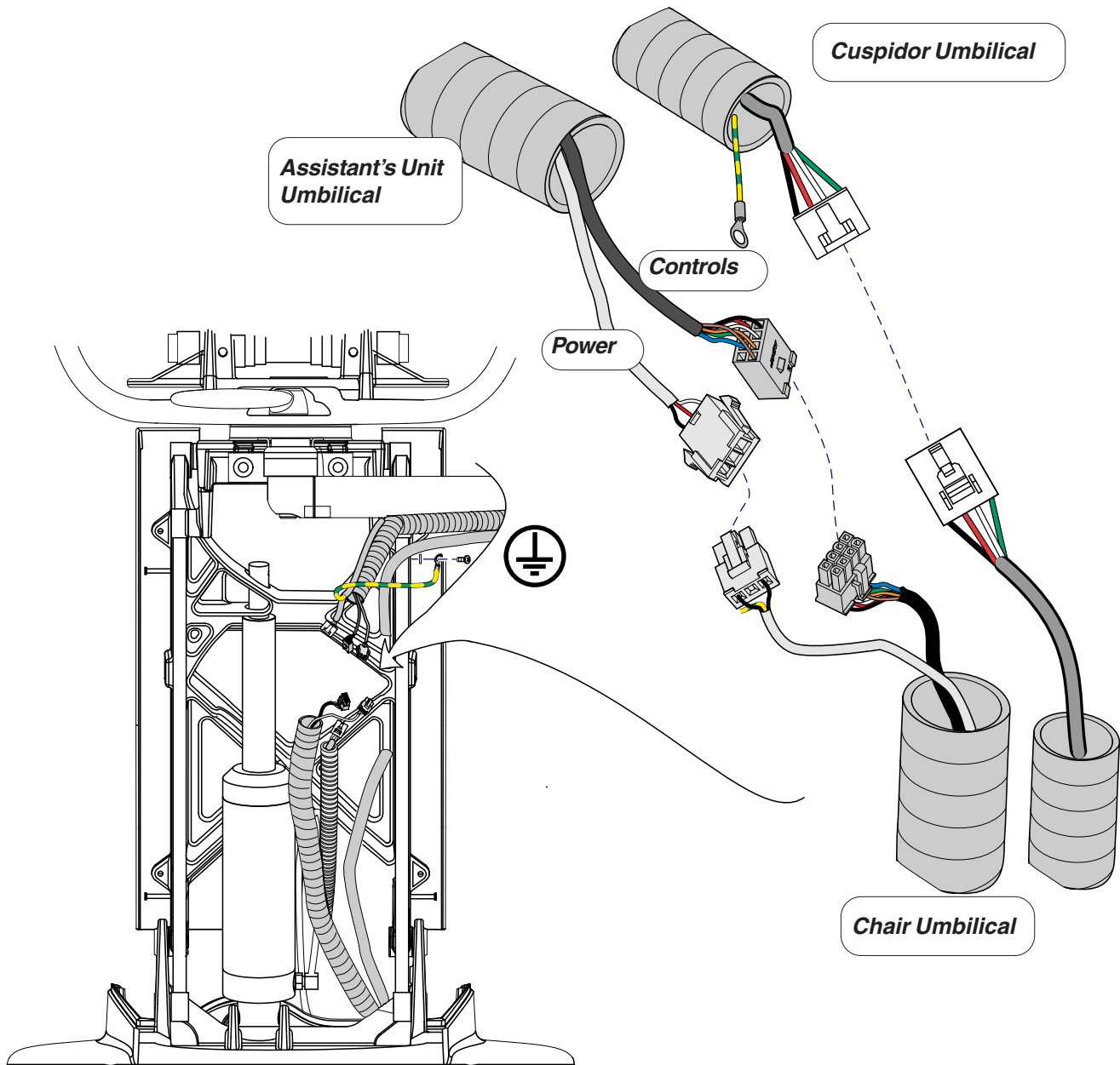


Step 13: Pull tubing/wiring toward extension plate to minimize slack. Secure with two cable ties .

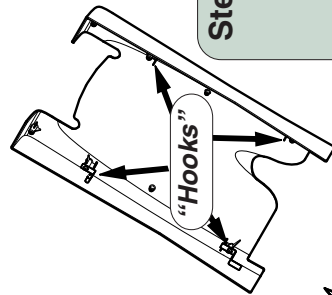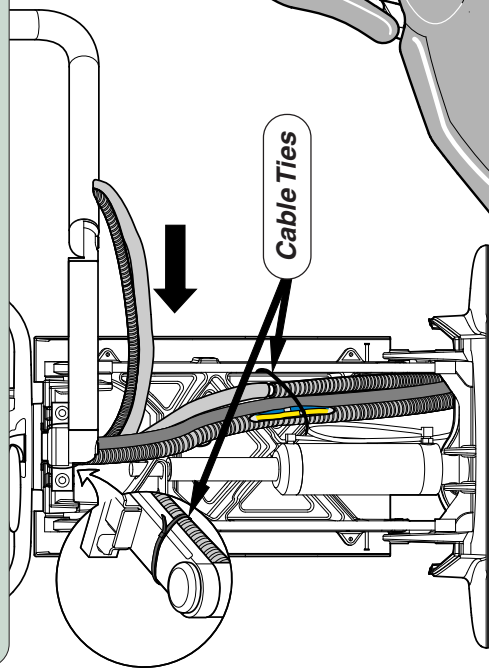


Step 14: Install safety bail cover.

Tip: Position bottom "hooks" on lower mounting pins. Align top "hooks" with upper mounting pins. Push cover Down, In, then pull Upward until upper "hooks" engage.

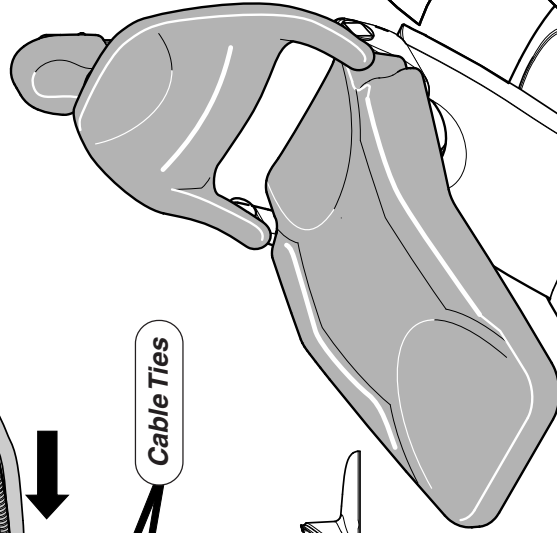

Tabs

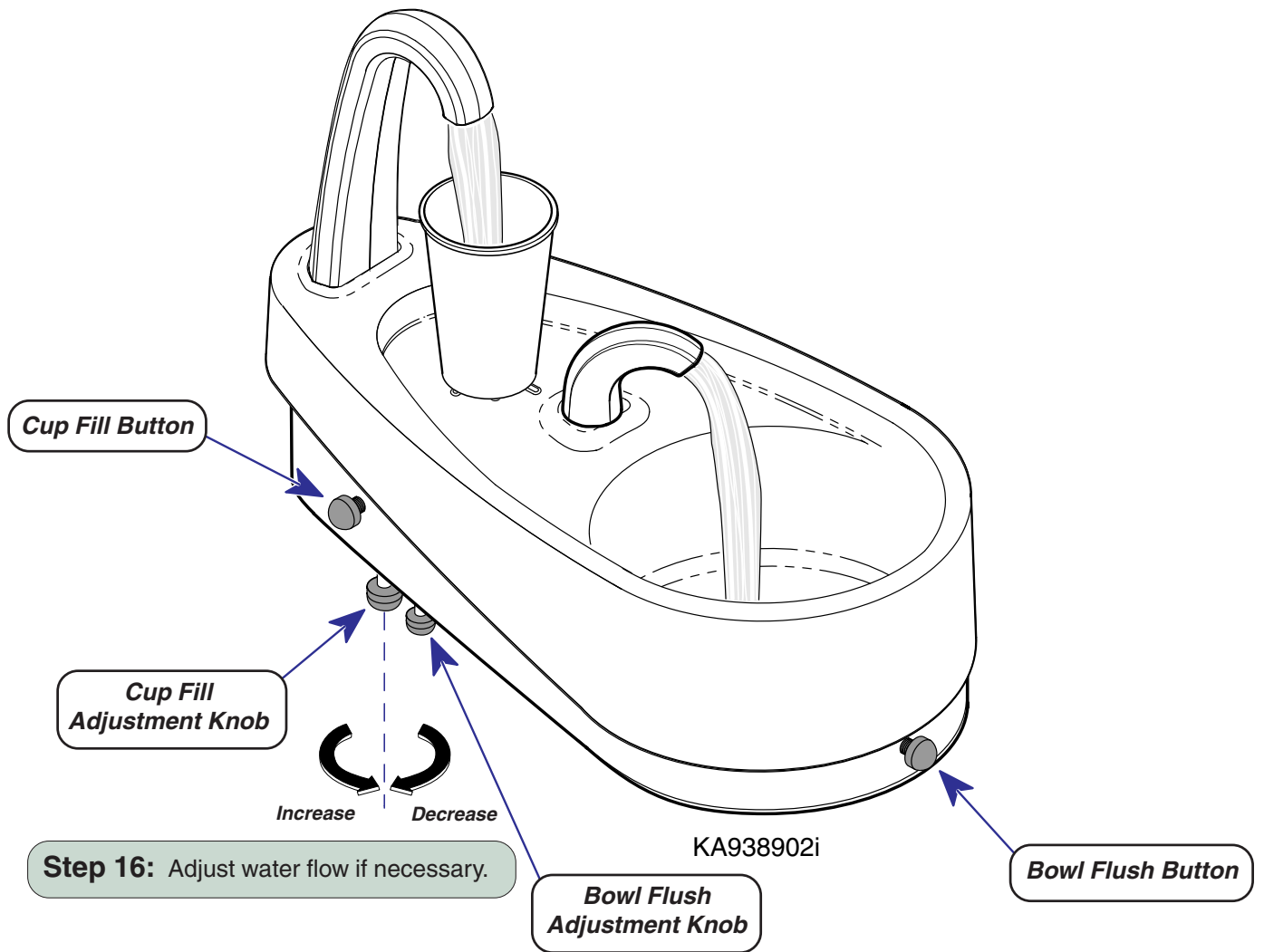

Midmark Corporation
 60 Vista Drive
 P.O. Box 286
 Versailles, OH 45380-0286
 937-526-3662
 Fax 937-526-5542
 www.midmark.com



Because we care.