

Touchpad Replacement Kit [002-0672-xx]



Components:


002-0672-01:
RH Touchpad 1

002-0672-02:
LH Touchpad 1



Special Tools:

none

For ProCenter Delivery Units:

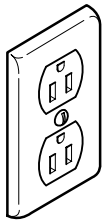


ProCenter Delivery Units
Step 1: Disconnect power to operator.

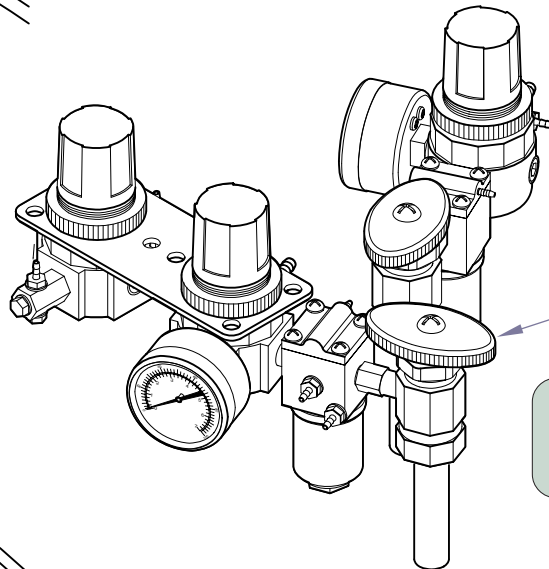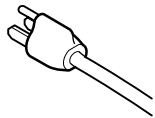

**Air Manual
Shut-Off Valve**

ProCenter Delivery Units
Step 2: Turn air manual shut-off valve
(in junction box) OFF.

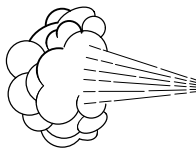

Air Button

ProCenter Delivery Units **Equipment Alert**




Residual air pressure *will* eject the plunger from the master ON/OFF switch.

Step 3: Press the air button on the syringe(s), hold button until all residual air pressure is expelled from the system.


DA162800i

For ProCenter Delivery Units - continued:

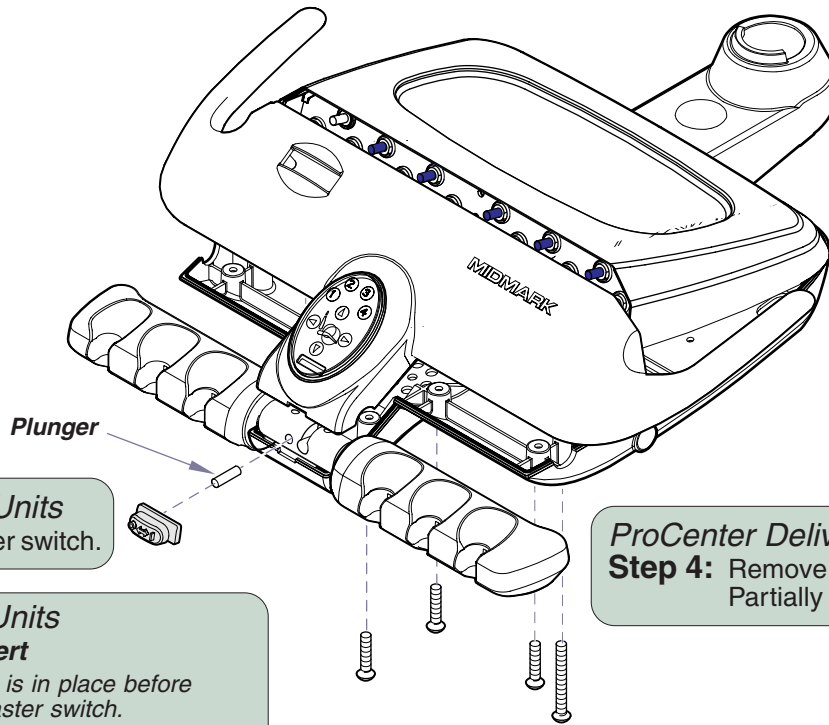


ProCenter Delivery Units
Step 5: Remove master switch.

ProCenter Delivery Units
Step 4: Remove eight screws.
Partially separate front cover.

ProCenter Delivery Units
Equipment Alert
 Be sure plunger is in place before installing the master switch.

Step 8: Position master switch & front cover.
Secure cover with eight screws.

ProCenter Delivery Units
Step 9: Turn air manual shut-off valve ON.
Connect power to operator.
Check for proper operation.

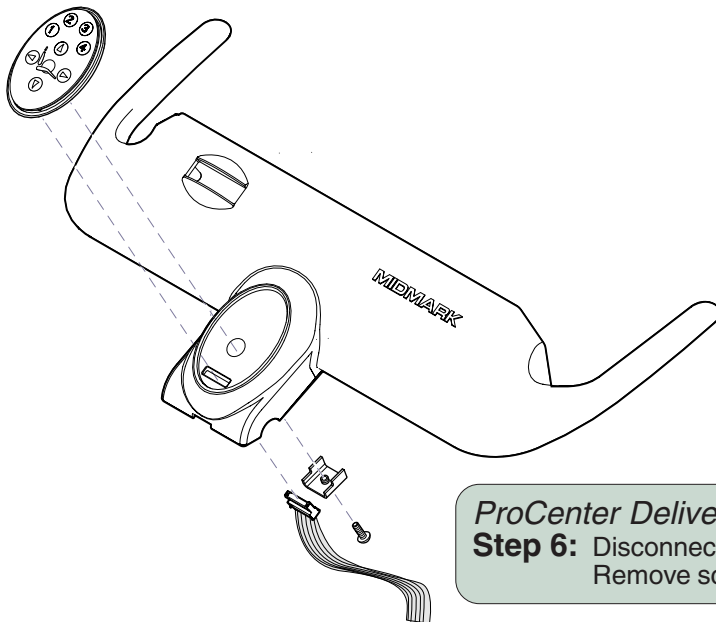


ProCenter Delivery Units
Step 6: Disconnect ribbon connector from touchpad.
Remove screw, bracket & touchpad.

ProCenter Delivery Units
Step 7: Position touchpad, then secure with bracket & screw.
Connect ribbon connector to touchpad.


KA975302i

For Assistant's Unit:

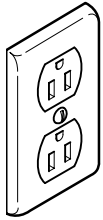


Assistant's Units


Step 1: Disconnect power to operatory.

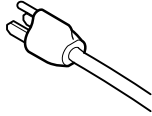

ProCenter Units

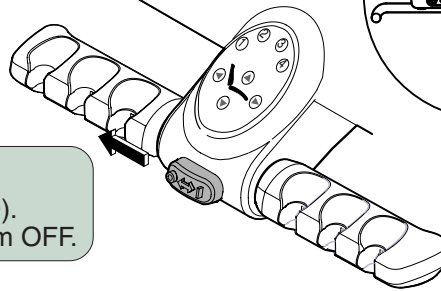


Whip-style Units

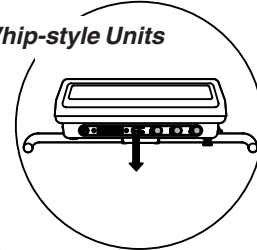

Assistant's Units

Step 2: Turn master switch OFF (O).
Turn facility vacuum system OFF.

DA162900i

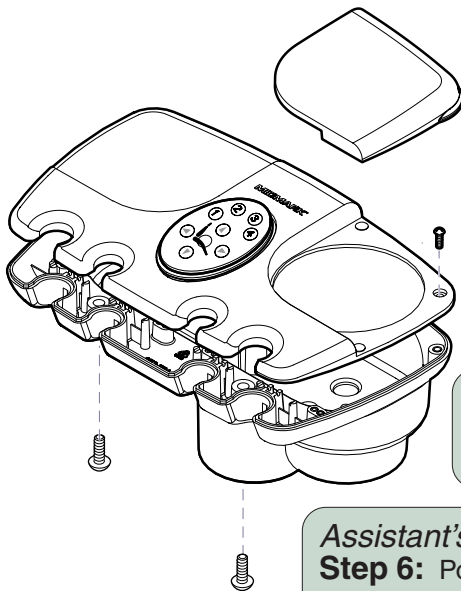

Assistant's Units

Step 3: Remove five screws, then partially separate top cover.

Assistant's Units

Step 6: Position top cover, then secure with five screws.

ProCenter Delivery Units


Step 7: Turn master switch ON.
Turn facility vacuum system ON.
Connect power to operatory.
Check for proper operation.

Assistant's Units

Step 4: Disconnect ribbon connector from touchpad.
Remove screw, bracket & touchpad.


Assistant's Units

Step 5: Position touchpad, then secure with bracket & screw.
Connect ribbon connector to touchpad.

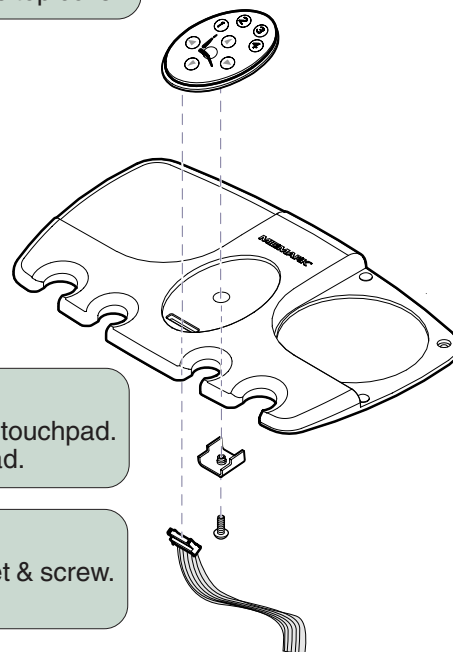


KA975402i

For Midmark Dental Chairs:

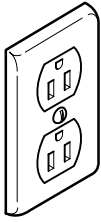

Midmark Dental Chair
Step 1: Disconnect power to chair.

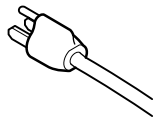



Midmark Dental Chair
Step 6: Connect power to chair.
Check for proper operation.

Midmark Dental Chair
Step 2: Remove mounting screw.



Midmark Dental Chair
Step 5: Install mounting screw.

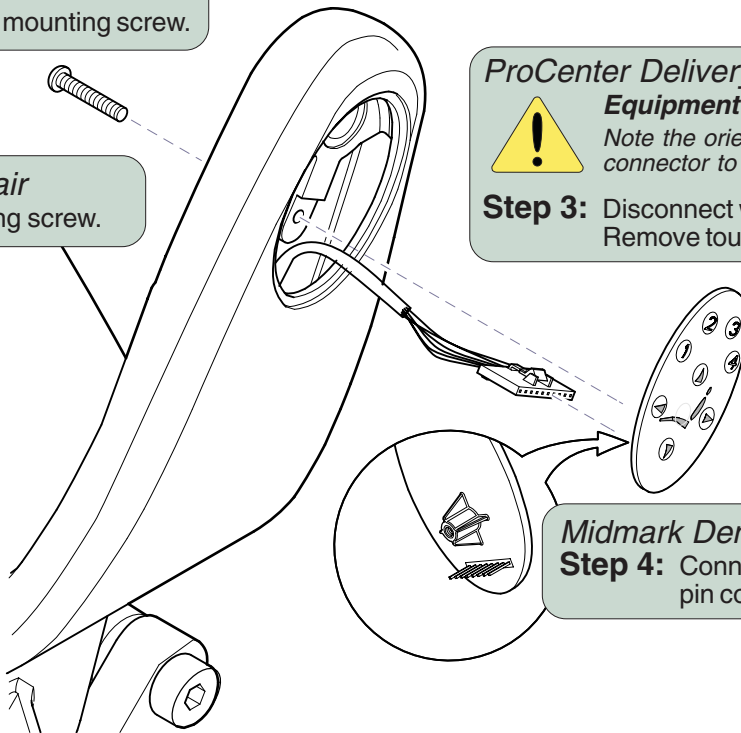

ProCenter Delivery Units
Equipment Alert
 Note the orientation of wire harness connector to touchpad pin connector.

Step 3: Disconnect wire harness.
Remove touchpad.

Midmark Dental Chair
Step 4: Connect wire harness to touchpad pin connector in proper orientation.

KA953702i