

Assistant's Heated Syringe Kit: 12:00 Units [002-0767-00]




Parts Bag Contents


- Heated Syringe
w/ tubing 1
- Wire Harness 1
- Cable Ties 3

Special Tools


none

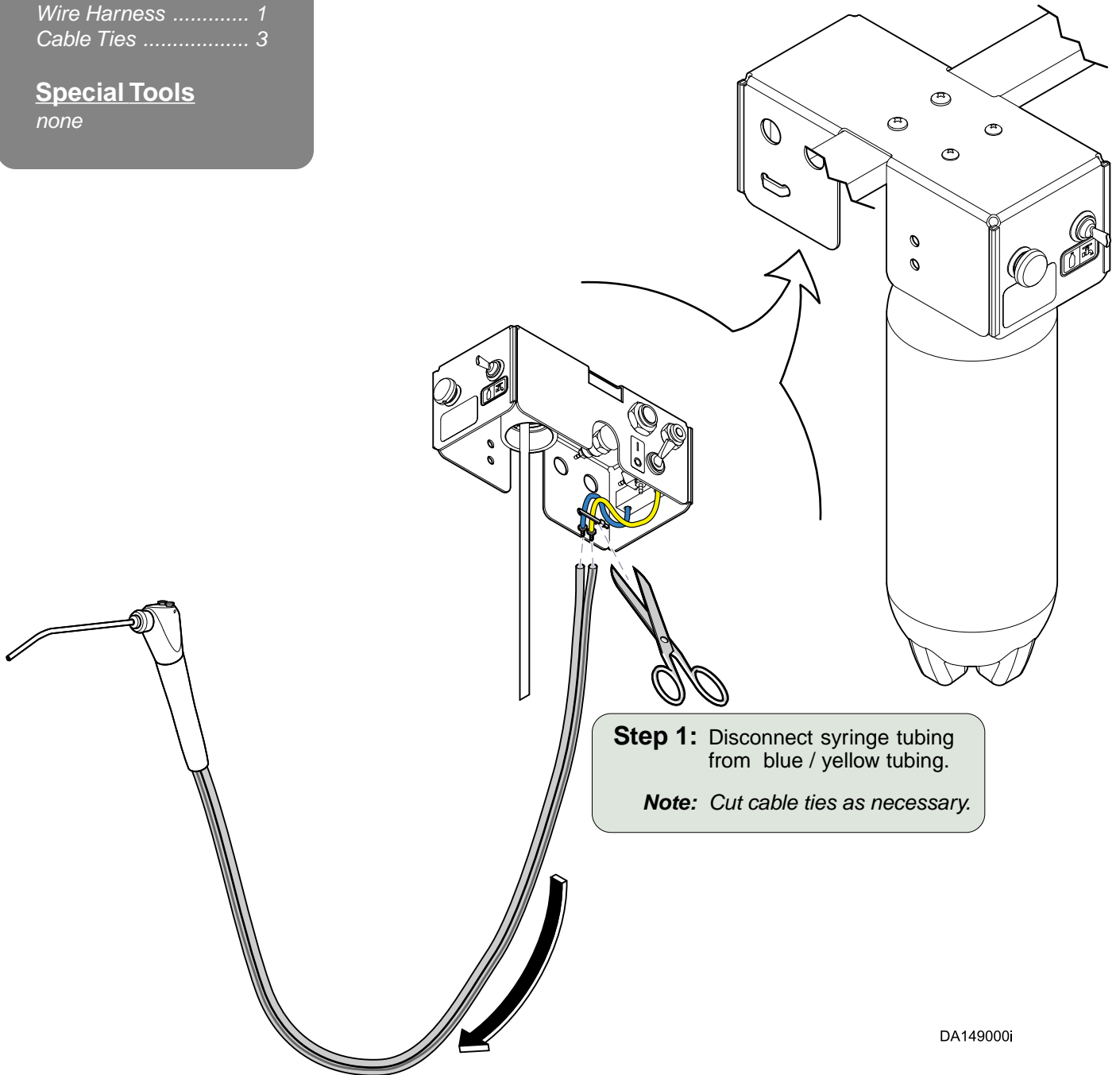

DA149000i

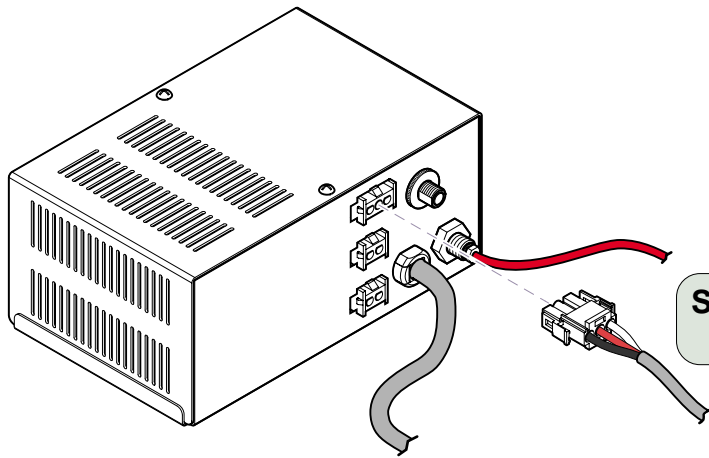


Step 2: Connect wire harness to the universal power supply.

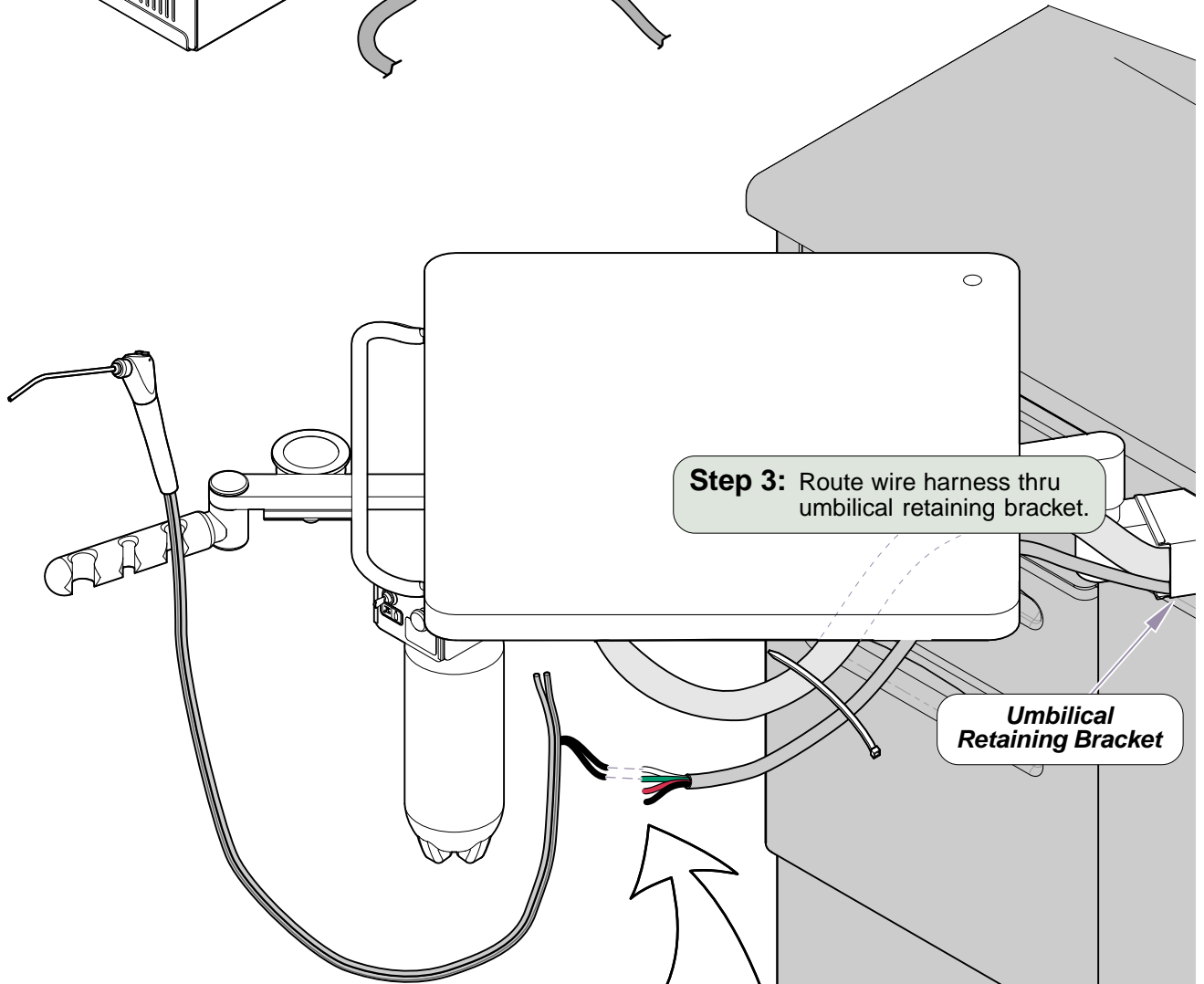

Step 3: Route wire harness thru umbilical retaining bracket.

Umbilical Retaining Bracket

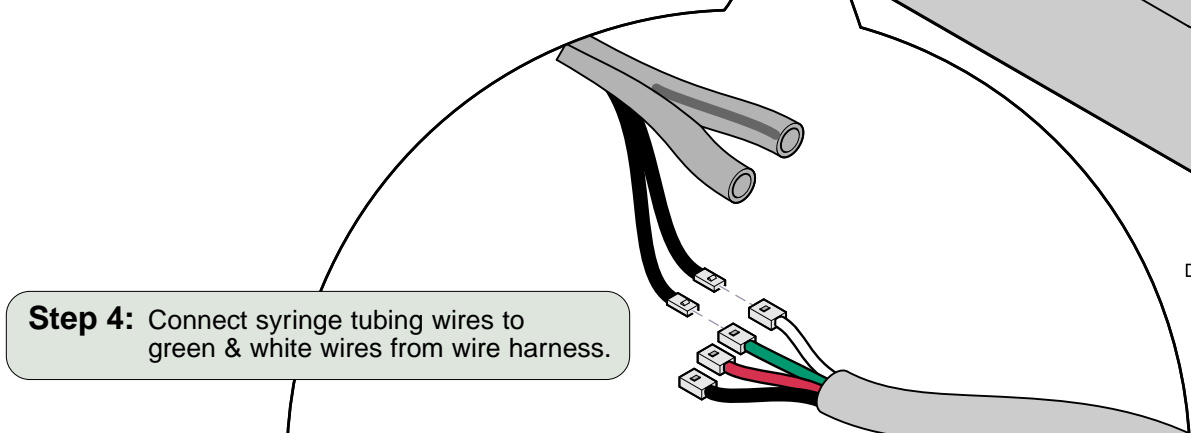

Step 4: Connect syringe tubing wires to green & white wires from wire harness.

DA149100i



Equipment Alert

Do not shorten heated syringe wires. The entire length is required for proper operation.
[Resistance measured across Molex connector must be 10 ± 1 ohms @ 77°F (25°C)]

