



Procenter Unit Master Switch Kit [002-1032-00]

- Components:**
 Master Switch 1
 and Gasket 1
 Instrument Holder
 Connector 1
 Air Valve 1
- No Special Tools**

Equipment Alert
 Turn OFF the Manual Air Shutoff Valve (in Junction Box).
 Purge any residual air in system.

- Step 1: Replace Instrument Holder Connector.**
- A) Remove screws and lift off Front Cover.
 - B) Remove two screws and the Instrument Holder Assembly from Bottom Housing.
 - C) Disconnect red and purple tubing from Air Valve.
 - D) Loosen two set screws and remove the Instrument Holder Connector.
 - E) Install new Instrument Holder Connector and tighten the set screws.
 - F) Install Instrument Holder Assembly onto bottom housing.
 - G) Connect red and purple tubing to the Air Valve and install the Master Switch.
 - H) Install Front Cover and install and Gasket for Master Switch.

