



255 LED Light Driver Board Installation [002-1891-00]

Components:
Driver Board

Special Tools:
none

Language of origin: English


 **Warning**
Always disconnect power before removing covers.
Failure to do so may result in personal injury.

 **Equipment Alert**
Do not damage harnesses when removing cover.

Step 1: Access driver board.
A) Turn light OFF and disconnect power.
B) Remove cover and components from back of lighthouse assembly shown.



Replace Driver Board

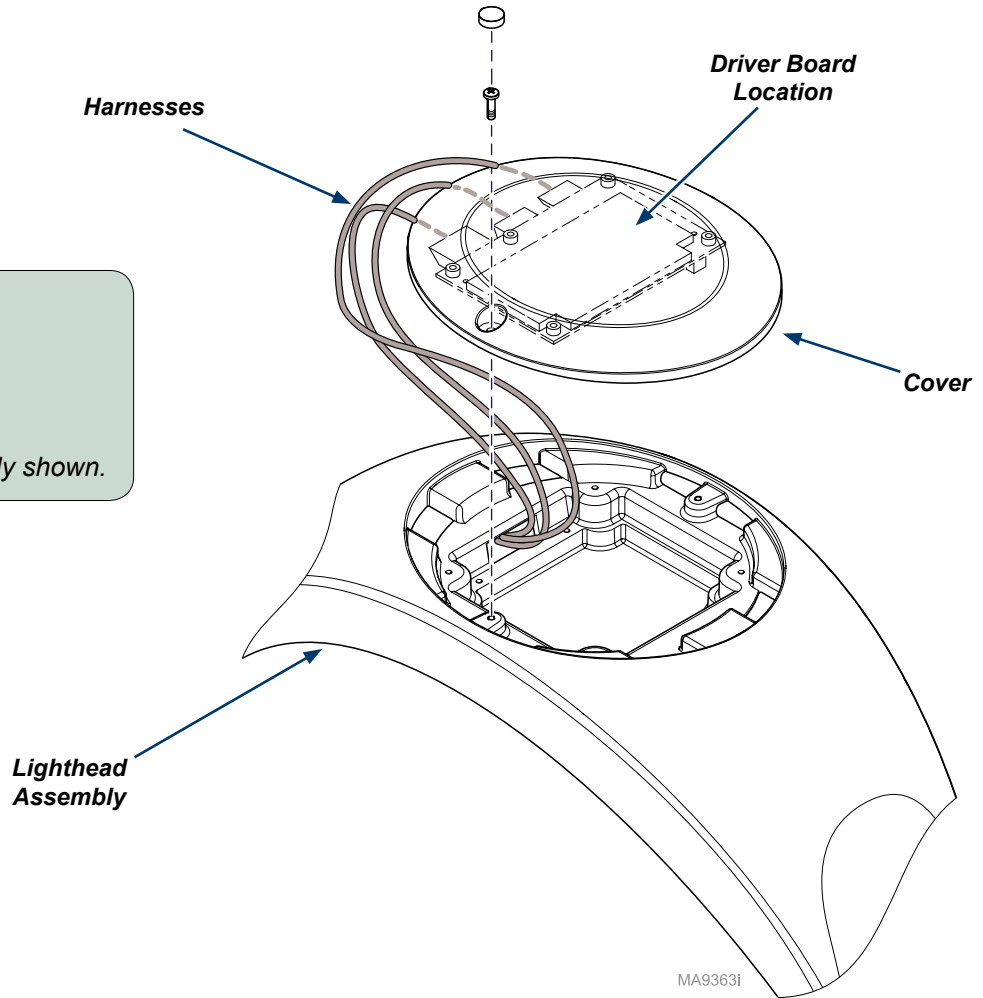

Step 2: Remove driver board.

A) Remove connectors and screws from cover shown.



Step 3: Install new driver board.

- A) Peel off film and secure driver board.
- B) Connect wire harnesses.
- C) Complete steps 1 through 2 in reverse order.
- D) Connect power and verify operation.