


**153672-002 Bien Air Curing Light
(12:00 FSC) (includes items 1 thru 22)**

Item	Description	Qty.
1	• Curing Light Box Assembly (includes items 2 thru 22)	1
2	•• Curing Light Kit	1
3	•• Velcro (loop)	2
4	•• Velcro (hook)	2
5	•• Tubing Connector (4-Port)	1
6	•• Socket Terminal	2
7	•• Receptacle (2 Circuit)	1
8	•• Accessories Harness	1
9	•• Check Valve	1
10	•• Fitting (1/8" Barb Tee)	2
11	•• Uni-Clamp	2
12	•• Barb Fitting (1/8")	2
13	•• Gasket	2
14	•• Fitting (1/16" x 1/8" Barb Tee)	1
15	•• Wiring Connector	2
16	•• Tubing (1/4" OD x 1/8" ID Clear)	AR
17	•• Tubing (1/8" OD Clear)	AR
18	•• 4-Hole Asepsis Handpiece Tubing	AR
19	•• Black Wire (Reference Tubing Diagram)	AR
20	•• Red Wire (Reference Tubing Diagram)	AR
21	•• Blue Wire (Reference Tubing Diagram)	AR
22	•• Brown Wire (Reference Tubing Diagram)	AR

Serial Numbers / Dates: all

DA1630001

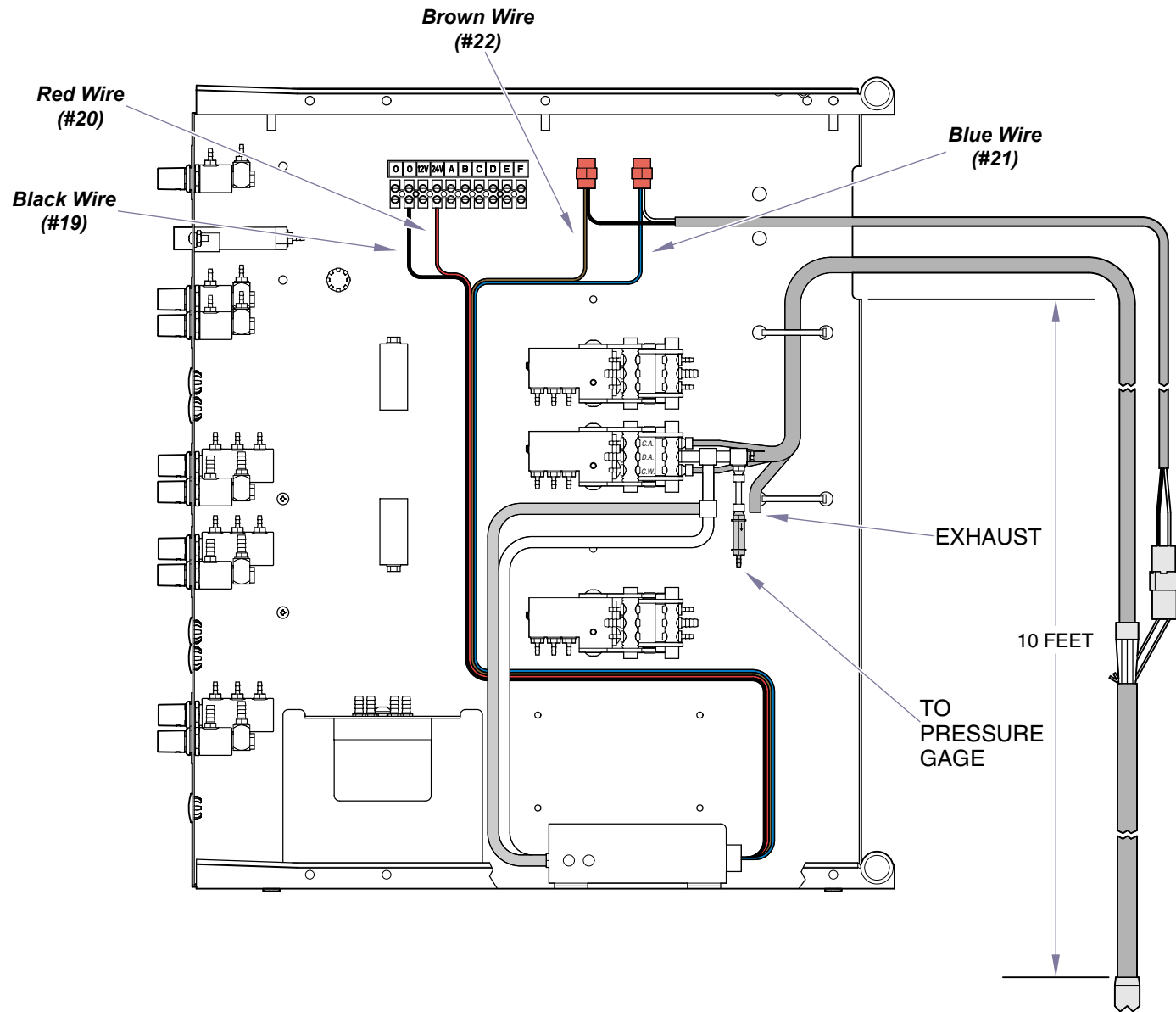

Serial Numbers / Dates:	all	
--------------------------------	-----	--

DA2028i