

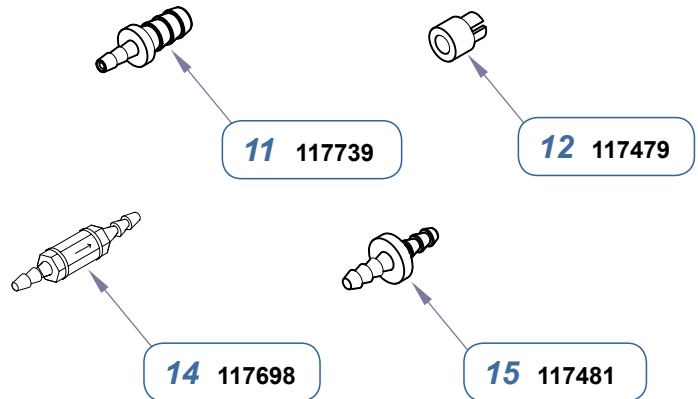


1 029-2854-00

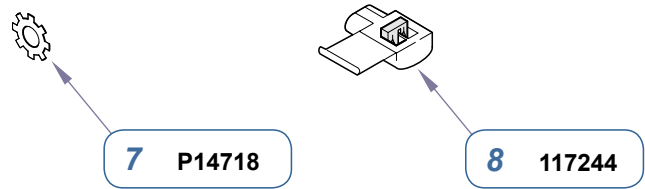

19 015-1482-00

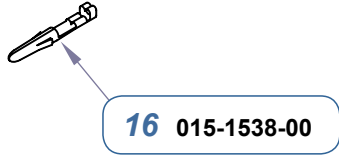


153631-002 Amdent Scaler System
(includes items 1)

Item	Description	Qty.
1	• 12:00 FSC Unit Scaler Assembly (includes items 2 thru 19)	Ref
2	•• Scaler Components	1
3	•• Scaler Mount Bracket	1
4	•• Screw (M3 x 5 x 5MM)	2
5	•• Scaler Cover	1
6	•• Screw (#6-32 x 3/8)	4
7	•• Lockwasher	1
8	•• Wire Connector	2
9	•• Tubing, (1/8" OD Clear)	AR
10	•• Tubing, (1/4" OD x 1/8" ID Clear)	AR
11	•• Barb Fitting, (1/16" to 1/8")	1
12	•• Uni-Clamp	4
13	•• Fitting, (1/16" Barb Tee)	1
14	•• Check Valve	1
15	•• Barb Fitting (1/16" to 1/16")	1
16	•• Socket Terminal	2
17	•• 2-Circuit Receptacle	1
18	•• Accessories Harness	1
19	•• Scaler Wiring (not Shown)	1

DA163600I

Serial Numbers / Dates: all

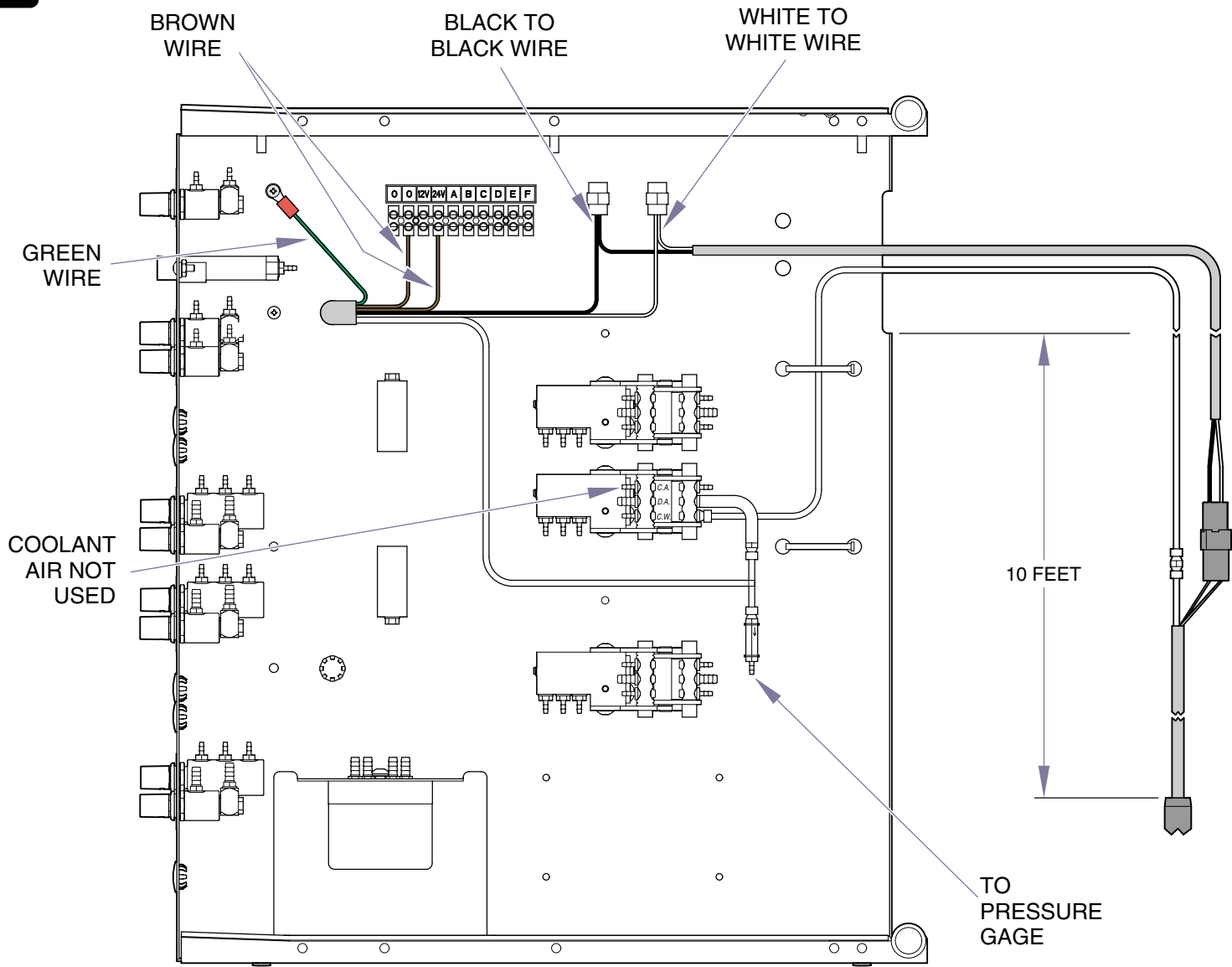

DA2026i

Serial Numbers / Dates:	<i>all</i>	
--------------------------------	------------	--