

**Note**  
Route handpiece tubing through slot in cover center.

**153767-004 Dual Digital Electric Handpiece [ProCenter] System**  
(includes Items 1 - 15)

| Item | Description  | Qty. |
|------|--|------|
| 1    | • Bien Air Digital Electric Handpiece Kit (includes Items 2 - 6) | 1    |
| 2    | • • Transformer (24 V - 32 V)                                    | 1    |
| 3    | • • Console  | 1    |
| 4    | • • Screw  | 4    |
| 5    | • • Board  | 1    |
| 6    | • • Relay Board  | 1    |
| 7    | • Electric Handpiece Mounting Bracket                            | 1    |
| 8    | • Screw (#6-32 x 3/8")   | 4    |
| 9    | • Digital Handpiece Cover  | 1    |
| 10   | • Electric Handpiece Bracket                                     | 1    |
| 11   | • Screw (M3 x 0.5 x 5 mm)  | 4    |
| 12   | • Standoff   | 1    |
| 13   | • EMI Suppression Core   | 1    |
| 14   | • Velcro (loop)  | 1    |
| 15   | • Velcro (hook)  | 1    |

**Serial Numbers / Dates:** V352593 thru Present




Connect Opaque 1/8" OD Tube to Coolant Air

Connect Clear 3/16" OD Tube to Drive Air

Connect Green 1/8" OD Tube to Coolant Water

Orange Connector to Relay Board - Route Wires thru EMI Core

Tee to Purple Line of Motor 2 Kink Valve  
 Connect Air Switch Wire to Slot 3 on PC Board  
 Tee to Purple Line of Motor 1 Kink Valve  
 Connect Air Switch Wire to Slot 2 on PC Board

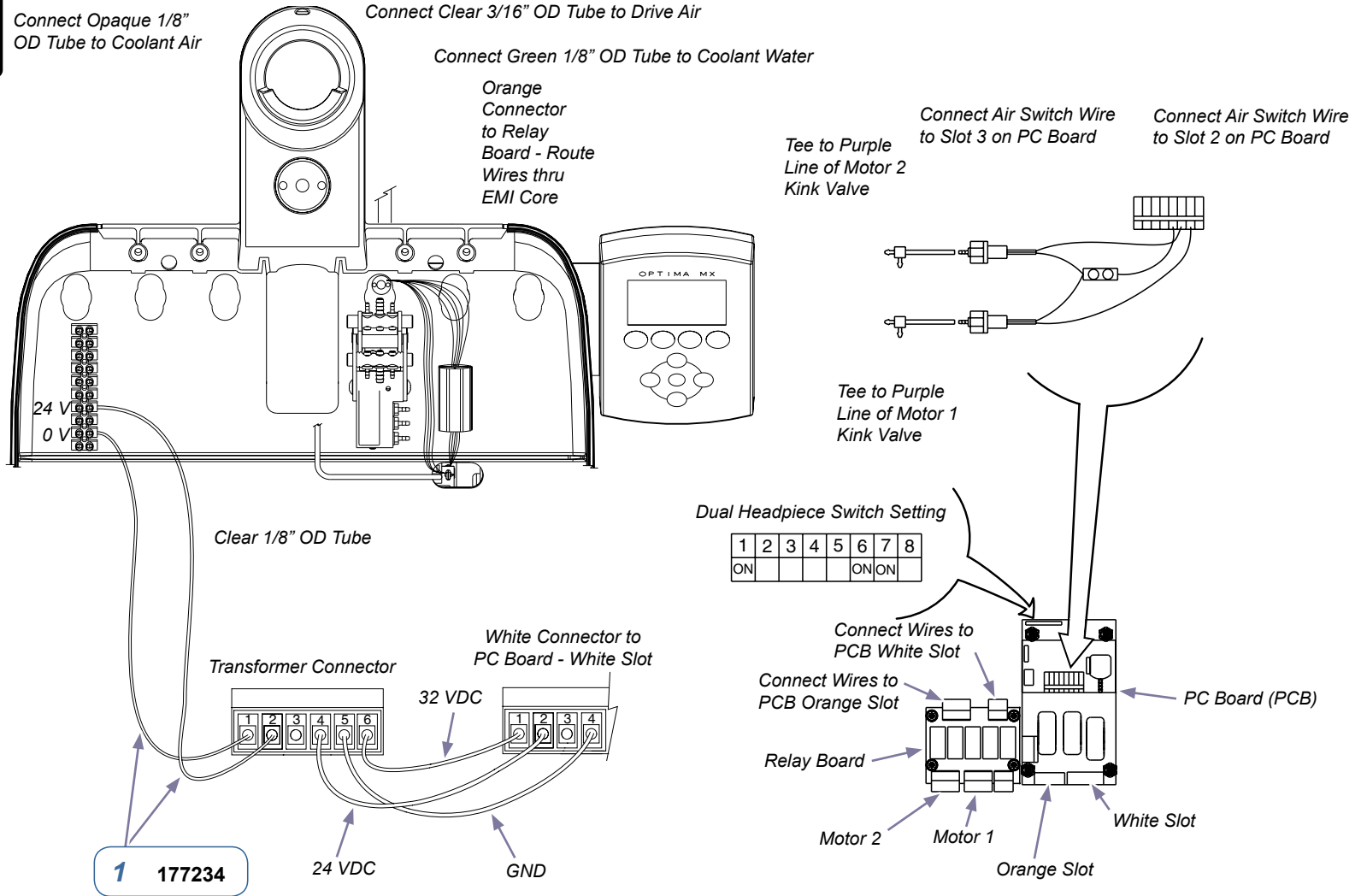


**Note**  
 Connections shown outside of head for clarity only.  
 Connections occur under head and concealed by cover.

DA2627I

**153767-004 Dual Digital Electric Handpiece [ProCenter] System (continued)**  
 (includes item 1)

| Item | Description                 | Qty.  |
|------|-----------------------------|-------|
| 1    | • Wire, Black (22 AWG 300V) | 5 ft. |

|                                |                      |
|--------------------------------|----------------------|
| <b>Serial Numbers / Dates:</b> | V352593 thru Present |
|--------------------------------|----------------------|

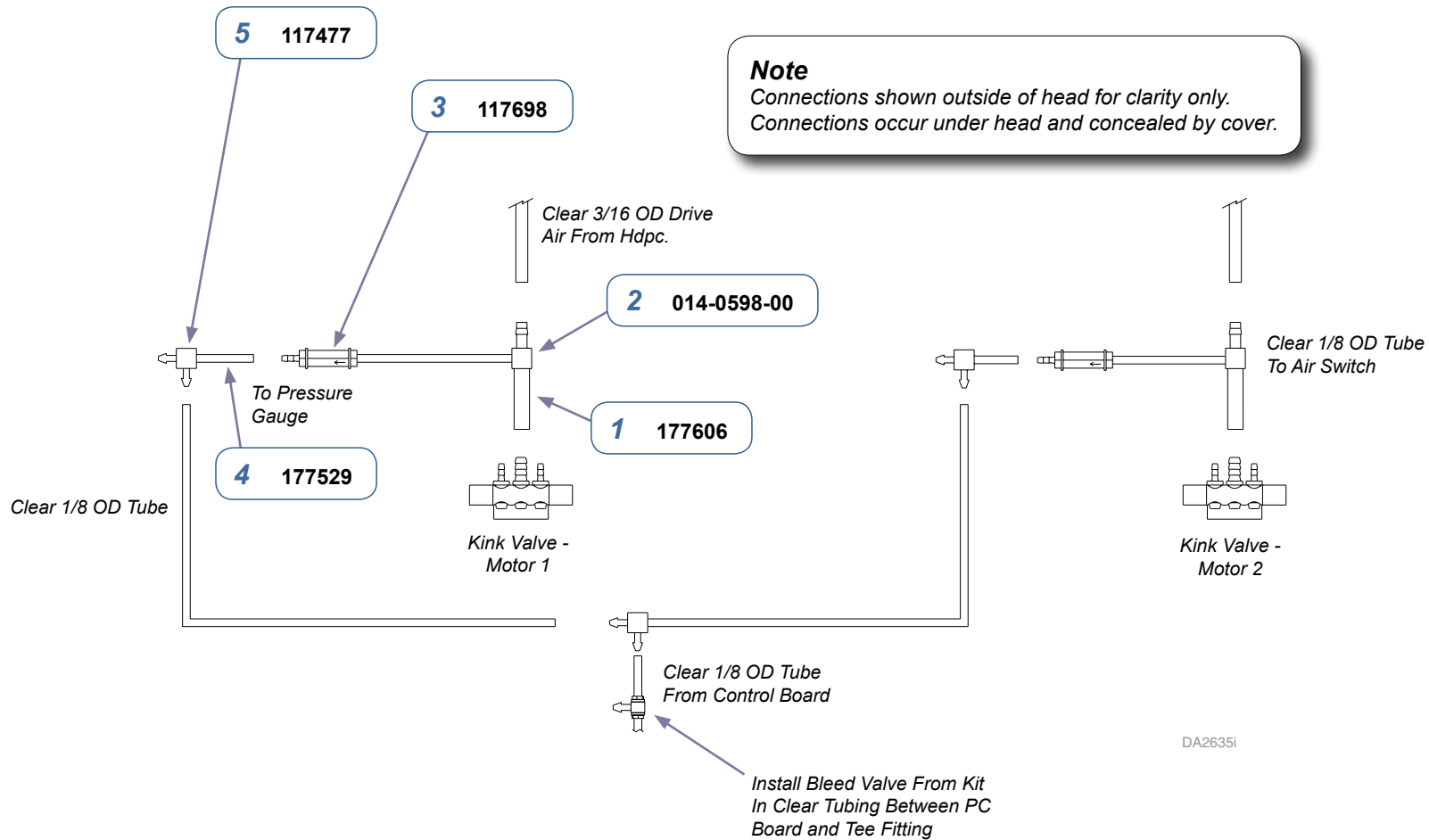

**153767-004 Dual Digital Electric Handpiece [ProCenter] System (continued)**  
(includes items 1 - 5)

| Item | Description               | Qty. |
|------|---------------------------|------|
| 1    | • Tubing, Clear           | AR   |
| 2    | • Fitting, Barbed Tee     | 2    |
| 3    | • Check Valve             | 2    |
| 4    | • Tubing, Clear (1/8" OD) | AR   |
| 5    | • Fitting, J              | 4    |

**Serial Numbers / Dates:** V352593 thru Present